

Product Data Sheet

miro-brand description	<p style="text-align: right;">miro-castlonguette <i>Orthopaedic splint system</i></p> <p>miro-castlonguette is an orthopaedic cast system which can be used for functional and partially immobilizing casts. Areas of application include fractures, tendon or capsula injuries, distortions. The splint system can also be used for the support of soft tissue injuries.</p> <p>miro-castlonguette consists of a multi-layered fibreglass system which is covered by a water-activated polyurethane resin. Between skin and fibreglass is a non-woven padding material for patients' comfort.</p> <p>The desired lengths of the splint will be cut off from a 4.5m roll which supports a cost effective treatment. The splint is breathable which enables air and moisture to circulate.</p>
-------------------------------	--



Measurements

Item-N°	Size	Price	Dispatch unit
13300	5,0cm x 4,5m		1 roll
13301	7,5cm x 4,5m		1 roll
13302	10,0cm x 4,5m		1 roll
13303	12,5cm x 4,5m		1 roll

Composition of Goods

- Cover material:** Non-Woven made of polyethylen-elasthan threads
- Middle layer:** fibreglass, 8 layers, impregnated with polyurethane resin
- Padding material:** Non-Woven made of polypropylen
- Air permeable
 - Water resistant when completely dry
 - Durable
 - X-Ray translucent

Technical details

- Bandage should be immersed in water at 20 °C – 25 °C
- Bandage will set in approx. 3-4 min.
- Weight-bearing: after 20 min.
- Shelf life: 3 years from production date
- Wearing time: up to 60 days
- Store opened rolls in a cool and dry place

Storage Requirements:

Storage: Do not store extremely warm (over 25 °C) or cold (under 10 °C)
Avoid storage next to radiators or outer gates.
„First in - first out“.

Stretchable (in %)	Sterilization	Washable	CE (on single product)	Custom-Tariff-N°
---	---	---	yes	9021.1090
Shelf Life	Classification	Annex (EU-guideline 93/42)		Rule
3 years	I	I and VII		---