


Product Data Sheet

miro-brand description	miro-polycast	<i>Polyester Cast Tape</i>
	<p>miro-polycast is a strong, easy-to-use cast tape made of a polyester fabric impregnated with polyurethane resin with a smooth finish. It is suitable for fractures, post-operative immobilization as well as for severe sprains and strains. Especially used for circular cast bandages and functional cast therapy. Polycast conforms well to contours providing a comfortable fit. Finished casts are very lightweight, durable and translucent to X-rays. The porous weave makes it air-permeable. Individually sealed in a foil laminate pouch. Gloves should always be worn when handling miro-polycast.</p> <p>Available in white, red or blue</p> <p style="text-align: right;">length: 3,6m</p>	

Measurements

Item-N°	Size	Price	Dispatch unit
12005	5,0cm x 3,6m		10 pcs.
12008	7,5cm x 3,6m		10 pcs.
12010	10,0cm x 3,6m		10 pcs.
12105	5,0cm x 3,6m		10 pcs.
12108	7,5cm x 3,6m		10 pcs.
12110	10,0cm x 3,6m		10 pcs.
12205	5,0cm x 3,6m		10 pcs.
12208	7,5cm x 3,6m		10 pcs.
12210	10,0cm x 3,6m		10 pcs.

Composition of Goods

Polyester fabric with polyurethane resin	- Air permeable
Polyester: 41%	- Water resistant when completely dry
Polyurethane: 52,5%	- Durable
Silikone: 3,2%	- X-Ray translucent
Catalysator: 3,3%	- 3 – 4 layers can be cut with scissors

Technical details

- Bandage should be immersed in water at 21 °C – 24 °C
- Immerse approx. 3-5 seconds, squeeze several times
- Bandage will set in approx. 3-5 min.
- Weight-bearing: after 20 min.
- Store unopened rolls in a cool dry place
- Colours differences may occur when humid and will disappear after drying-up

Storage Requirements:

Storage: Do not store extremely warm (over 25°C) or cold (under 10°C)
Avoid storage next to radiators or outer gates.
„First in - first out“.

Stretchable (in %)	Sterilization	Washable	CE (on single product)	Custom-Tariff-N°
---	---	---	yes	9021.1090
Shelf Life	Classification	Annex (EU-guideline 93/42)		Rule
3 years	I	I and VII		---